**A Device Generic Name**

Injectable Dermal Filler

**B Description**

MONALISA is a sterile, transparent gel of stabilized hyaluronic acid of non-animal origin. MONALISA is supplied in a glass syringe with a luer-lock fitting. The contents of the syringe have been sterilized using moist heat. The product is for single use only. Disposable sterile needles are provided with syringe. Information about size of the needle is printed on its packaging.

**C Intended Use**

MONALISA Filler is intended to be used for soft tissue augmentation and restoration of facial lipoatrophy in patients over the age of 21.

**D Indication for use**

1. Precautions Before Use  
- Use the product after examining the packaging for damage or contamination of the product and checking the expiration date. Do not use expired/damaged, or contaminated products.

- The patient must be informed of the purpose, expected results, precautions, and possible adverse events before treatment.  
- Proper informed consent is essential to obtain favorable results. Injection technique of the manufacturer prior to initial injection is important for a more successful injection.

- Only qualified medical professionals must administer the treatment.

## 2. Treatment Procedure

- The use of sterile needles and cannula matching the Luer-lock specifications of the syringe is essential for safety and efficacy.

- The needle must be correctly attached to the syringe for safety. Incorrect attachment can result in the detachment of the syringe and the needle. Hold the junction of the syringe and the Luer-lock with one hand and the sterile needle shield with the other.

- Rotate the needle to securely fit to the syringe.  
- The treatment area is to be disinfected with aseptic solution.

- Do not bend the needle to prevent breakage.

- The product should be injected into the dermis or the lower part of the dermis.

- To remove air from the interior of the syringe, press the rod slowly until there is a small drop at the tip of the needle.

- Aspirate before use is recommended when using a needle. Inject content while slowly pulling the needle backwards.

- Stop injection before fully pulling out the needle.

- Use a new needle and cannula for each injection site.

- Massage the injection site following injection to facilitate conformation to surrounding tissue.

- Periodic additional treatment helps to achieve desired results. The treatment can be adjusted depending on the patient's needs and the preference of the patient.

3. Storage Following Use  
- Dispose of used and over products including needles and canulas after procedure.

- Due to possible risks of contamination, distortion, and infection, left over products are not to be reused.

- The reuse of this product, a single use medical device, is prohibited.

**E Precautions During Use**

## 1. Warning

- This product is to be used only for intradermal injections.

- This product is not to be injected into blood vessels.

- Unintended introduction into blood vessels could cause occlusion and thrombosis.

- When injected into blood vessel, this may result in serious side effect, such as blindness, it is suggested not to use this filler around middle of forehead having a thin skin and a high probability that injected into blood vessels. So pay special attention when operating.

- Stop injection if the injection site becomes pale, and massage the treatment site until it returns to a normal color.

- The product is not to be used on patients with bleeding disorders or those taking thrombolytics or anti-coagulants.

- This product is not to be re-sterilized.

- This product is not to be mixed with other products before injection.

2. Precautions Regarding Age, Sex, and Physical Conditions

- Do not use product into an area where another manufacturer's product is present. Do not inject the product into areas where non-injectable implants are already present.

3. Storage Following Use  
- Dispose of used and over products including needles and canulas after procedure.

- Due to possible risks of contamination, distortion, and infection, left over products are not to be reused.

- The reuse of this product, a single use medical device, is prohibited.

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Medical device

Sterilized using steam heat

Single sterile barrier system with protective packing outside

Fragile, handle with care

**K Handling method**

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Luer-lock

Hold the junction of the syringe and the Luer-lock with one hand and the sterile needle shield with the other. Rotate the needle to securely fix it to the syringe.

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